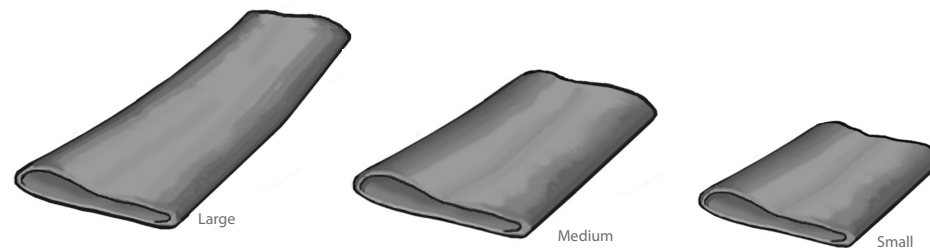
Manual - ReadySlide

Manual nr: 00817- En, Nord Rev. 5
Max 150 kg/330 lbs

Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Torshamnsgatan 35
SE-164 40 Kista
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299

Internet: www.handicare.com
E-mail: info@handicare.se

English

Svenska

Norsk

Dansk

ReadySlide is a thin, pliable and very easy-to-use sliding mat that facilitates various types of lying transfers of users, positioning in chairs and application of lifting slings.

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Functional inspection

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Kontrol af funktion

Visual inspection
Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

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Always read the instructions

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Les alltid bruksanvisningen

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Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product.

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Do not leave the patient unattended during a transfer situation.

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Note

Remember to remove the product after use, as it has a very low-friction surface.

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NOTE! Always lock the wheels on the bed, gurney and wheelchair.

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Placement/removal

To place ReadySlide under the patient, lean the patient in the opposite direction. Make use of the sheet to lean the patient. Push ReadySlide in under the patient as far as possible (1). To remove, pull on the lower end of ReadySlide(2).

English

Turning

Position ReadySlide, and then turn the patient by rolling him/her over on one side or by pulling on the sheet. The patient slides easily on ReadySlide. Remove ReadySlide.

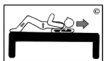
Bed to gurney

Position ReadySlide, move the gurney up beside the bed. Remember to lock the wheels. Push or pull the patient over to the gurney. Remove ReadySlide.

Higher up in bed

Place ReadySlide under the patient's shoulders. Ask the patient to bend the knees, lift the seat and push up. Also works well with the assistance of a caregiver. Remove ReadySlide.

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English

Transfer bed/wheelchair/toilet

Lock the wheelchair wheels and place ReadySlide under the patient's seat so that it is both on the bed and the wheelchair. Ask the patient to grasp the armrests and, using his or her own weight, slide over into the wheelchair. Raise the feet from the floor with the aid of a foot stool to reduce friction under the thighs. This transfer can also be performed by the caregiver with the aid of a care belt for added safety. Remove ReadySlide after completing the transfer.

For transfers between wheelchair and toilet, place ReadySlide as described above. By shifting his/her weight, the patient slides over to the toilet/wheelchair. Use a care belt if the patient requires the assistance of a caregiver. A foot stool also facilitates the transfer. Remove ReadySlide after completing the transfer.

Size (cm):

| | |
|-------|-----------------------|
| 11210 | 195 x 70 |
| 11215 | 195 x 70 with handles |
| 11230 | 110 x 70 |
| 11235 | 110 x 90 |
| 11240 | 50 x 60 |
| 11250 | 70 x 70 |

Material:

Polyamid

Care of the product:

Read the product label.

Do not use rinsing agent. To prolong product life, avoid tumble-drying.